

# Amazing SURE® Nursing Advisor System

Distributed by **TurunUK Ltd**  
Healthcare Solutions

## Monitor Patient Activity with Information Technology


The SURE System generates the following reports by default:



## Standard Reports


### Overview

Provides an aggregate view of when alerts are occurring daily, weekly, and by time of day. Insightful visualisation of peak times to aid in resource scheduling.

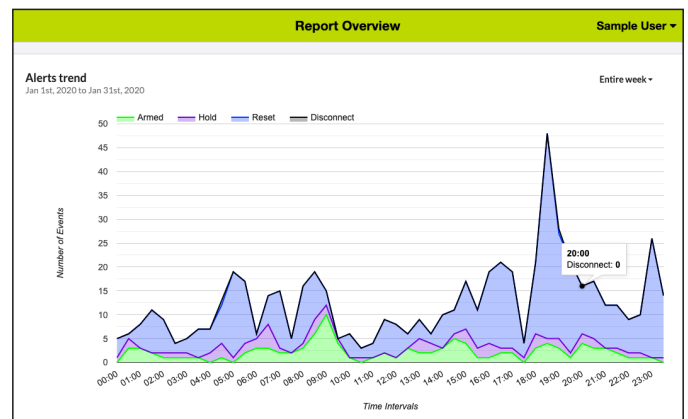

### Timeline

A minute-by-minute timeline of all devices for a single day. Displays every entry and exit to a monitored surface as well as the durations on those beds and chairs. This is useful to visualise a patient's past activities, especially for staff members just starting a shift.

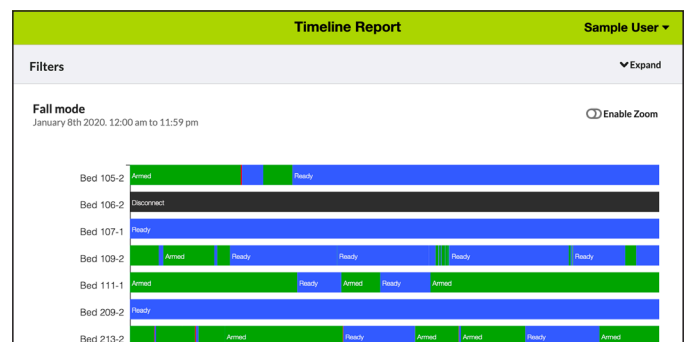

### Alert Log

An aggregate and detailed list of every area's alerts. Includes the alert detail such as duration, escalation path, staff responses, survey data, and more.

| Area Name | Total Events | Average Duration | Max Duration |
|-----------|--------------|------------------|--------------|
| Room 201  | 14           | 07s              | 10s          |

| Area     | Date       | Start             | End               | Duration | Resolution | Event Report |
|----------|------------|-------------------|-------------------|----------|------------|--------------|
| Room 201 | 03/29/2021 | 10:30:11 am (CDT) | 10:30:19 am (CDT) | 00:00:07 | reset      | 👁️ 📄         |
| Room 201 | 03/29/2021 | 10:50:14 am (CDT) | 10:50:24 am (CDT) | 00:00:09 | reset      | 👁️ 📄         |
| Room 201 | 03/29/2021 | 1:17:30 pm (CDT)  | 1:17:35 pm (CDT)  | 00:00:05 | reset      | 👁️ 📄         |
| Room 201 | 03/29/2021 | 2:49:19 pm (CDT)  | 2:49:25 pm (CDT)  | 00:00:06 | reset      | 👁️ 📄         |
| Room 201 | 03/29/2021 | 3:25:07 pm (CDT)  | 3:25:13 pm (CDT)  | 00:00:06 | reset      | 👁️ 📄         |
| Room 201 | 03/29/2021 | 5:09:44 pm (CDT)  | 5:09:54 pm (CDT)  | 00:00:09 | reset      | 👁️ 📄         |
| Room 201 | 03/29/2021 | 5:17:15 pm (CDT)  | 5:17:20 pm (CDT)  | 00:00:05 | reset      | 👁️ 📄         |

# Amazing SURE® Nursing Advisor System


## Dashboard

An at-a-glance view of the last 24 hours for staff members starting their shifts. Provides a recent summary to get staff up to speed on what has happened so that they can provide better care more quickly.

| Surface                   | Mode     | Current Time Occupied | Last 24 Hours<br>Total Time Occupied | Day                   |             | Night                 |             |
|---------------------------|----------|-----------------------|--------------------------------------|-----------------------|-------------|-----------------------|-------------|
|                           |          |                       |                                      | Longest Time Occupied | Total Exits | Longest Time Occupied | Total Exits |
| ● Attc 1                  | Activity |                       | 00m                                  | 0                     | 0           | 0                     |             |
| ● Attc 2                  | Activity |                       | 00m                                  | 0                     | 0           | 0                     |             |
| ● Attc 3                  | Activity |                       | 00m                                  | 0                     | 0           | 0                     |             |
| ● Back Conference         | Fall     |                       | 00m                                  | 0                     | 0           | 0                     |             |
| ● Hll 1                   | Fall     |                       | 00m                                  | 0                     | 0           | 0                     |             |
| ● Hll 2                   | Fall     |                       | 00m                                  | 0                     | 0           | 0                     |             |
| ● Hll 3                   | Fall     |                       | 00m                                  | 0                     | 0           | 0                     |             |
| ● Room 201                | Fall     | 24hrs 00m             | 24hrs 00m                            | 0                     | 24hrs 00m   | 0                     |             |
| ● Small Conf 1            | Fall     |                       | 00m                                  | 0                     | 0           | 0                     |             |
| ● Todd Above Matt Chair 1 | Activity |                       | 00m                                  | 0                     | 0           | 0                     |             |
| ● Todd Above Matt Chair 2 | Activity |                       | 00m                                  | 0                     | 0           | 0                     |             |
| ● Training Bed            | Fall     |                       | 00m                                  | 0                     | 0           | 0                     |             |
| ● Wheelchair              | Fall     |                       | 00m                                  | 0                     | 0           | 0                     |             |

## User Timeline

A timeline view of all facility staff members who have been logged into the SURE system.

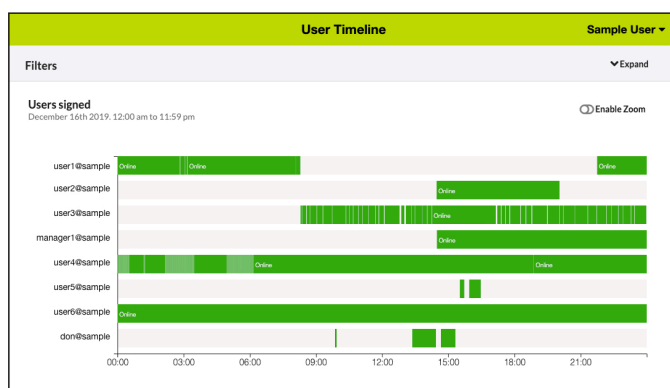

## More Features

### Data Management

The system can export data on patient activity; for example, from a hospital to an acute care facility, so patients can be given continuous attention on based on their activity.

### Stealth® Compatibility

The SURE system works with Stealth under-mattress pads, which offer their own advantages. Learn more and calculate your savings at [calc.stealthpads.co.uk](http://calc.stealthpads.co.uk).



Schedule a live demonstration with one of our staff!

020 7731 0132

turun.co.uk

Turun UK Ltd.

DOC # 22.87006C.11